


Admin manula

Carestream system

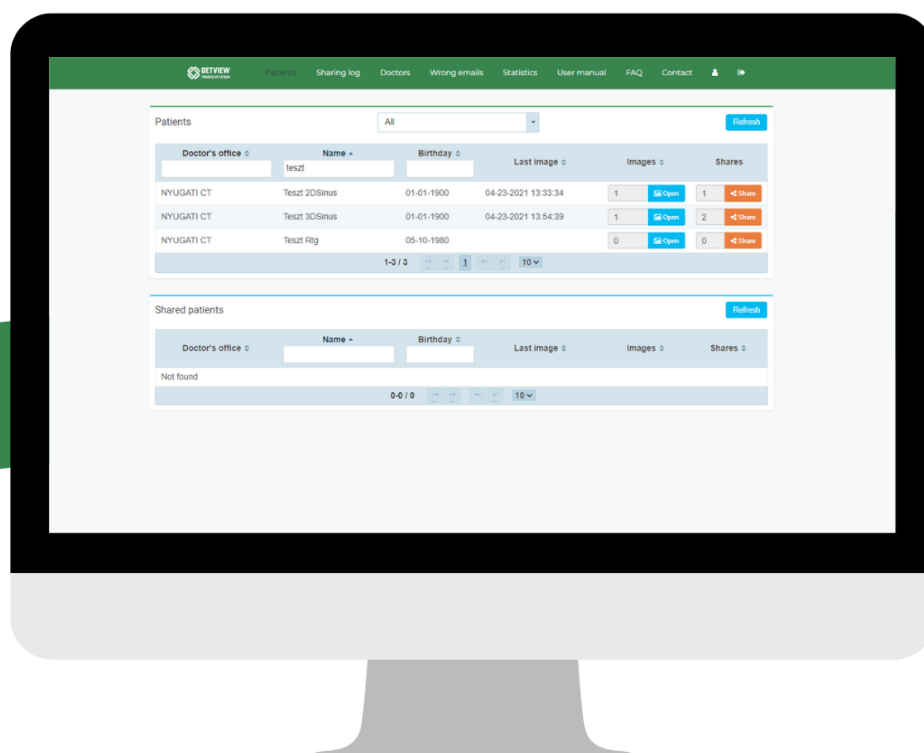

DetCom Systems Ltd.

Version: 4.0

© 2024

Content

Content 2

Introduction 2

Login page, Login 2

Doctors Page 4

Sharing log page..... 4

Patients Page 4

Wrong emails page 8

Statistics page 8

Profil page..... 8

Introduction


The document contains a detailed description of the admin level of the DetView system.

Login page, Login


There are additional language options available on the site, which can be selected from the drop-down menu.

Detailed description of the login:


1, You receive a message with accept link. If you clicking on this link, you agree to store your email address in the DetView system.

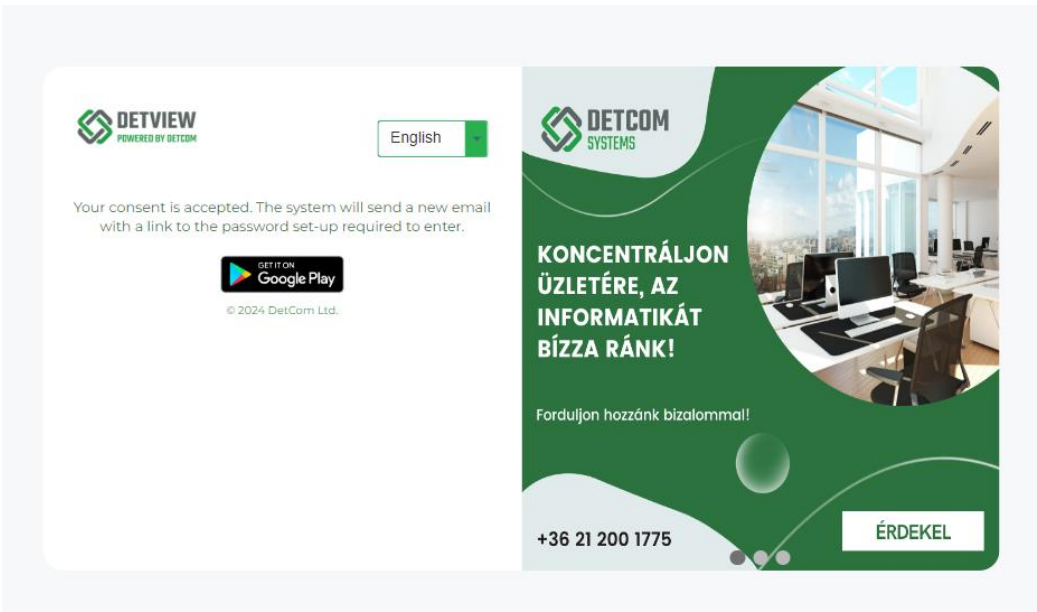

2, In the case of successful, you receive a new message with password setting link. Where you can enter own password. When you choose your password you should consider security, so it is advisable to choose a password in the following way: minimum 8 characters, including special characters, numbers, and capitalization.

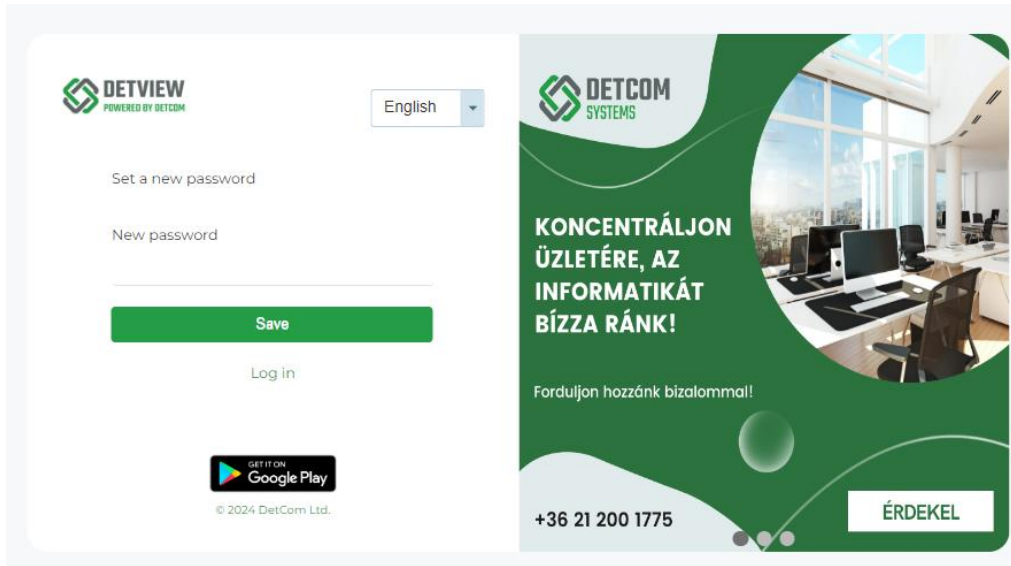


3, In the case of successful save, you receive email.

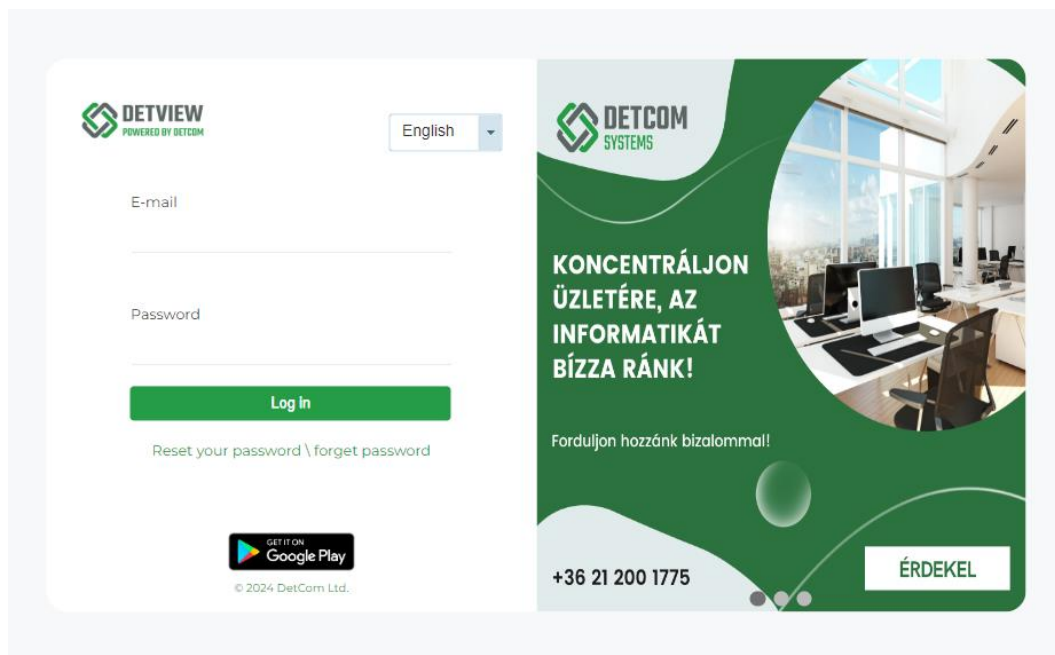

4, Then you get your main login surface. Enter your e-mail address and password here and click on **"Sign In"**

Once you have logged in, you can access the top menu to go to Patients, Sharing log, Doctors, User Manual, FAQ, Contact page, Profile and Logout icons.

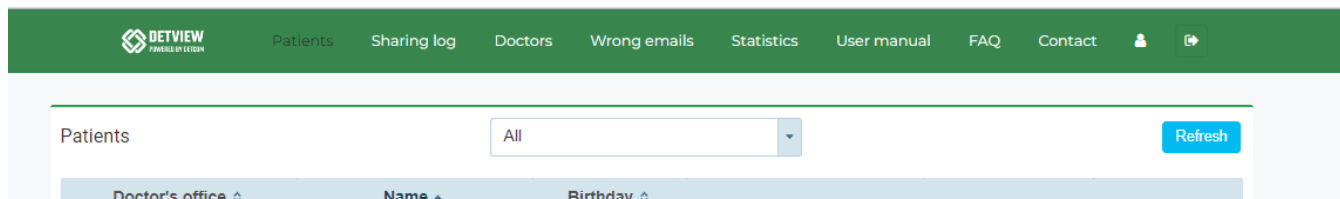


The screenshot shows the 'Patients' page in the DetView admin interface. At the top is a green navigation bar with the DetView logo and links to Patients, Sharing log, Doctors, Wrong emails, Statistics, User manual, FAQ, and Contact. Below the navigation bar is a search area with a text input containing 'Patients', a dropdown menu set to 'All', and a blue 'Refresh' button. Below this is a table header with columns: Doctor's office, Name, and Birthday.

Doctors Page

Here you can add or edit the email addresses of doctors, currently this function is not in use.

Sharing log page

In the sharing log you can view previously shared patients the sharing doctor with assigned doctor or patients and their dates with sharing states. The green tick indicates the active shares and the red dustbin deleted shares.

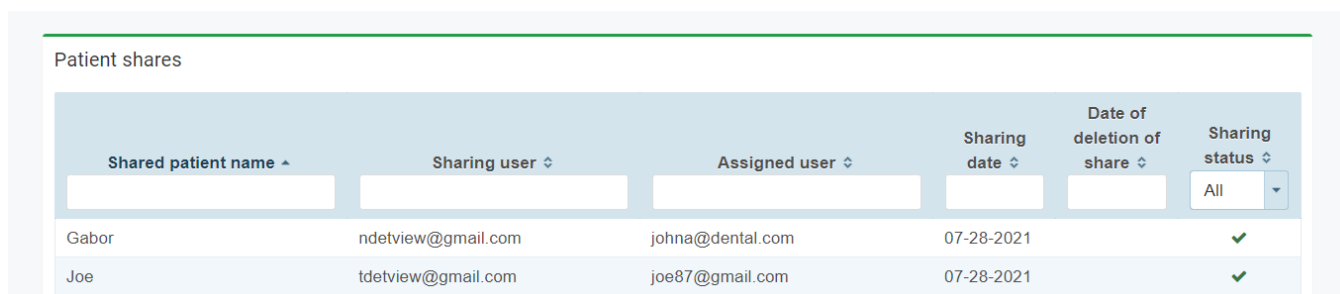




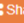



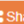


The screenshot shows the 'Patient shares' table in the DetView admin interface. The table has columns: Shared patient name, Sharing user, Assigned user, Sharing date, Date of deletion of share, and Sharing status. There are two rows of data, both with a green checkmark in the Sharing status column.



Shared patient name	Sharing user	Assigned user	Sharing date	Date of deletion of share	Sharing status
Gabor	ndetview@gmail.com	johna@dental.com	07-28-2021		✓
Joe	tdetview@gmail.com	joe87@gmail.com	07-28-2021		✓

Patients Page

On this page, you can download 2D and 3D images of patients assigned to the user. Here you also have the opportunity to share the patient's images with other doctors and patients.

From the top menu, you can access the User manual and if you need add-ons for the application, you can download them from here.



Patients						Refresh	
All							
Doctor's office	Name	Birthday	Last image	Images	Shares		
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 	0 		
Cdent	DemoX-ray 2	03-19-2003	05-17-2024 11:21:36	122 	0 		
NFRC	DemoX-ray 2	03-19-2003	08-07-2024 14:58:34	567 	0 		
NFRC	Patient(en)information 1	03-20-2003	08-08-2024 14:10:38	1914 	0 		
NFRC	teszt teszt	05-10-1980	10-18-2023 12:58:49	4 	0 		
1-5 / 5						10	

Shared patients						Refresh	
Doctor's office	Name	Birthday	Last image	Images	Shares		
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 	0 		
1-1 / 1						10	

Detailed description of the page:

There are two sections on the page, one with the patients and one with the shared patients. The latter show the patients who are shared with other doctors or patients

- Columns on the surface can be sorted as desired.
- Filling the "**Name**" and the "**Date of Birth**" columns can result in faster searches.
- By clicking on the "**Open**" button:

Patients						Refresh	
All							
Doctor's office	Name	Birthday	Last image	Images	Shares		
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 	0 		

we get all the images of the patient that can be retrieved from the system:

Time ↕	Image type ↕	Teeth ↕	
05-17-2024 15:01:51	Carestream 3D		Download without 3D Carestream Viewer Download with 3D Carestream Viewer
05-17-2024 14:56:35	Carestream 3D		Download without 3D Carestream Viewer Download with 3D Carestream Viewer
05-17-2024 14:56:31	Carestream 3D		Download without 3D Carestream Viewer Download with 3D Carestream Viewer
05-17-2024 14:56:27	Carestream 3D		Download without 3D Carestream Viewer Download with 3D Carestream Viewer
05-17-2024 14:56:11	Carestream PANO		Download without 2D Viewer Download with 2D Viewer
05-17-2024 14:56:10	Carestream PANO		Download without 2D Viewer Download with 2D Viewer

1-6 / 6 1 10

There are several ways to download recordings on this page:

1. Click on the "**Download without 2D Viewer**" button to download the image in "dcm" format.
2. Clicking on the "**Download without 2D Viewer**" button will immediately download the image in "dcm" format, which includes the X-ray viewer.
3. After clicking on the "**Download 3D without Cerastream Viewer**" button, the image will be downloaded in "3D" format, so you can load it into your own Cerastream or other X-ray viewer systems.
4. After clicking on "**Download with 3D Cerastream Viewer**", the image will be downloaded in "3D" format, which includes the X-ray viewer.
5. After clicking on "**Download without Cerastream Viewer**", the image will be downloaded in "PANO" format, so you can load it into your own Cerastream or other X-ray viewing systems.
6. After clicking on "**Download with Cerastream Viewer**", the image will be downloaded in "PANO" format, which includes the Cerastream viewer.

If we return to the previous page:


Patients

All

Refresh


Doctor's office ▾	Name ▲	Birthday ▾	Last image ▾	Images ▾	Shares
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 <div>Open</div>	0 <div>Share</div>

By clicking on the **"Share"** button, you can share the images of the patient with a doctor or patient.

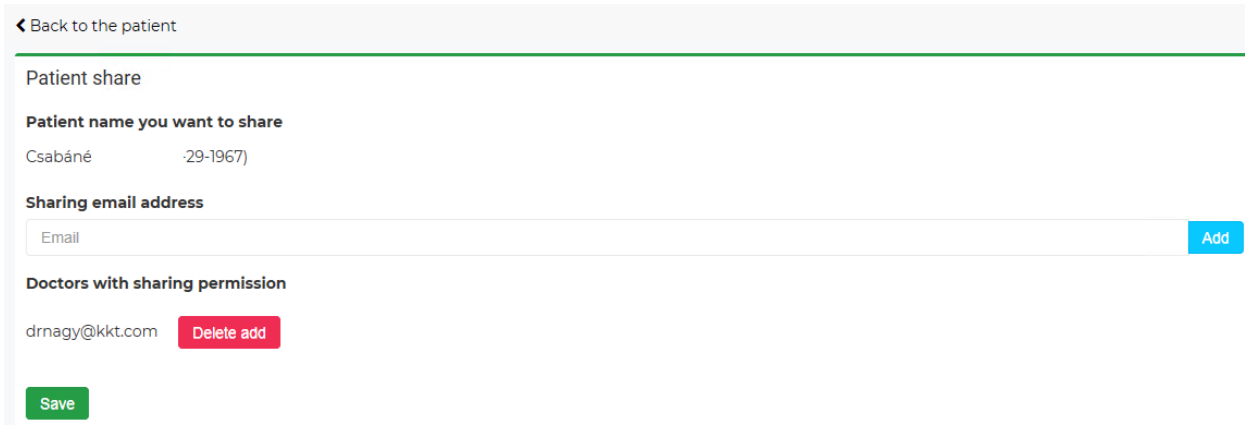

- Here enter the email address of the doctor or patient with whom we want to share the images.

First, select from the drop-down menu to share with a Patient or Doctor and enter the email address. After clicking on the **"Add"** button, a pop-up window will appear, where you will be informed about the data protection contribution. If you are impressed, click the **"Save"** button.

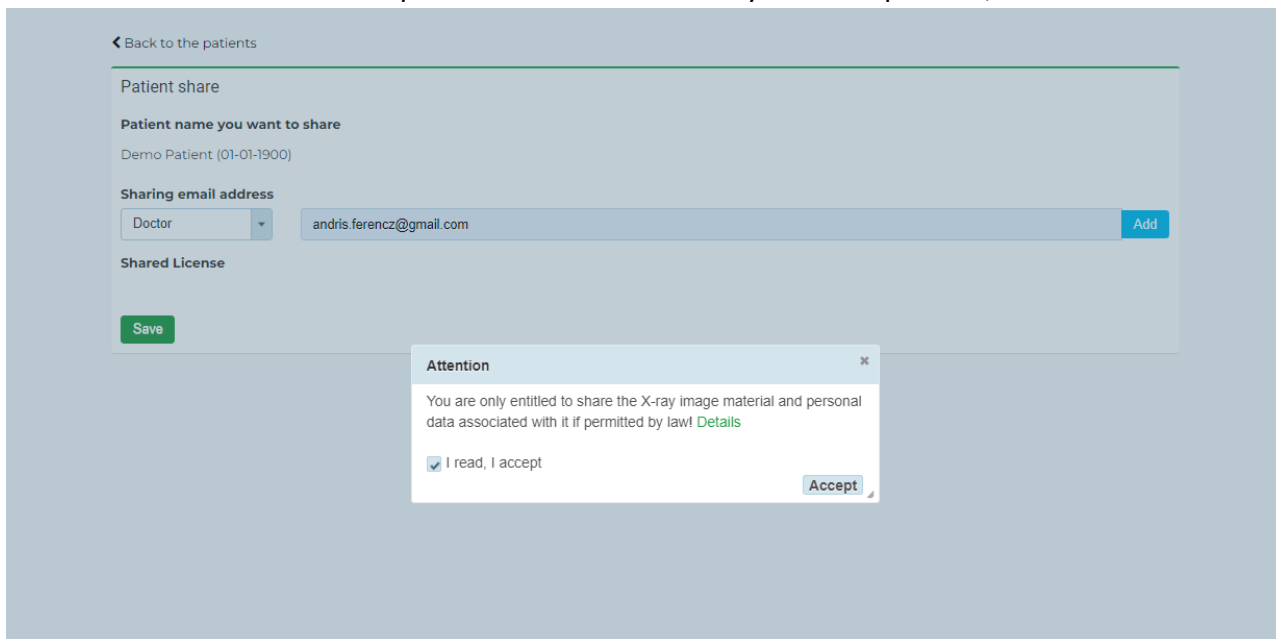


However, we also have the option for deleting added email addresses by clicking on the **"Delete add"** then **"Save"** button. In both cases, the system sends a notification notice to the given Email addresses.

- By default, only 10 patients and 10 images can be seen on the pages but it can be increased to 50 by using the drop-down menu below. If you would like to see more, you can turn a page on the bottom of the page.

- By clicking on "**Sign out**", which is on the right, you can log out of the system.

Wrong emails page

Here you can see the email addresses that the system was unable to send because they are not real email addresses or have simply been recorded incorrectly. This is a notification interface to correct the error.

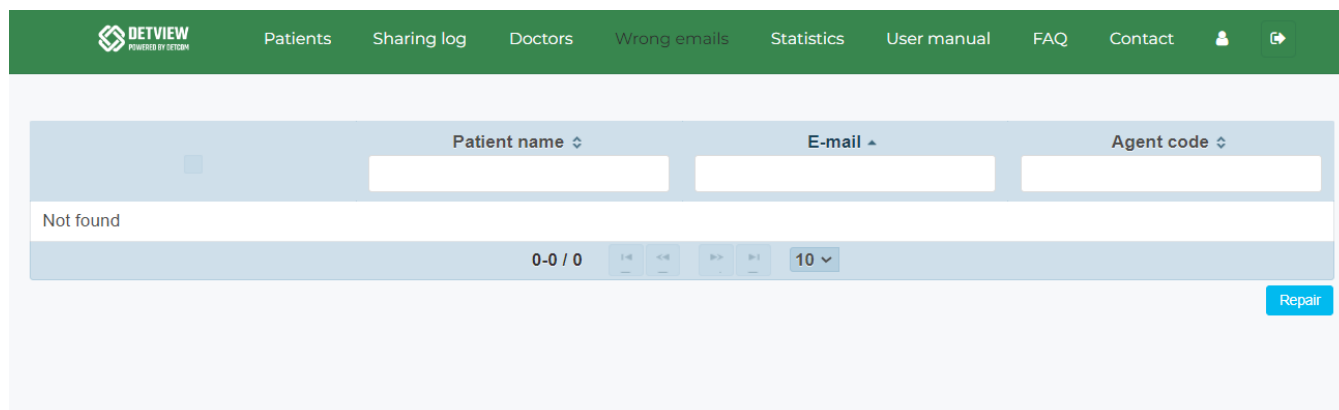

Statistics page

On this page you can view your entries and downloads. Who downloaded what, when and how many times.

Filters

07-09-2024

08-09-2024

Download all data

Download to Excel

Download logins to excel

Downloads

Agent code	E-mail	Downloads2	Tipus	Orvos neve
ICT	health@dent.me	5	3D Recording	Dr. Aenni
ICT	y.hegy@gmail.com	1	Panorama Recording	Dr. Geer
ICT	balmaz@gmail.com	11	Panorama Recording	Dr. Ross E
ICT	info@dental.hu	12	Panorama Recording	Dr. Biden

Profil page

In the profile menu you have the option to user to change their password or edit their details, including manage individual settings.

Password change**Personal data****E-mail address:** detviewtest@detcom.hu**Name** 

I don't want to be notified about new products, promotions and gift coupons.

You can initiate the permanent deletion of your user account by sending a message to helpdesk@detcom.hu.