


User manula

Sirona - Sidexis system

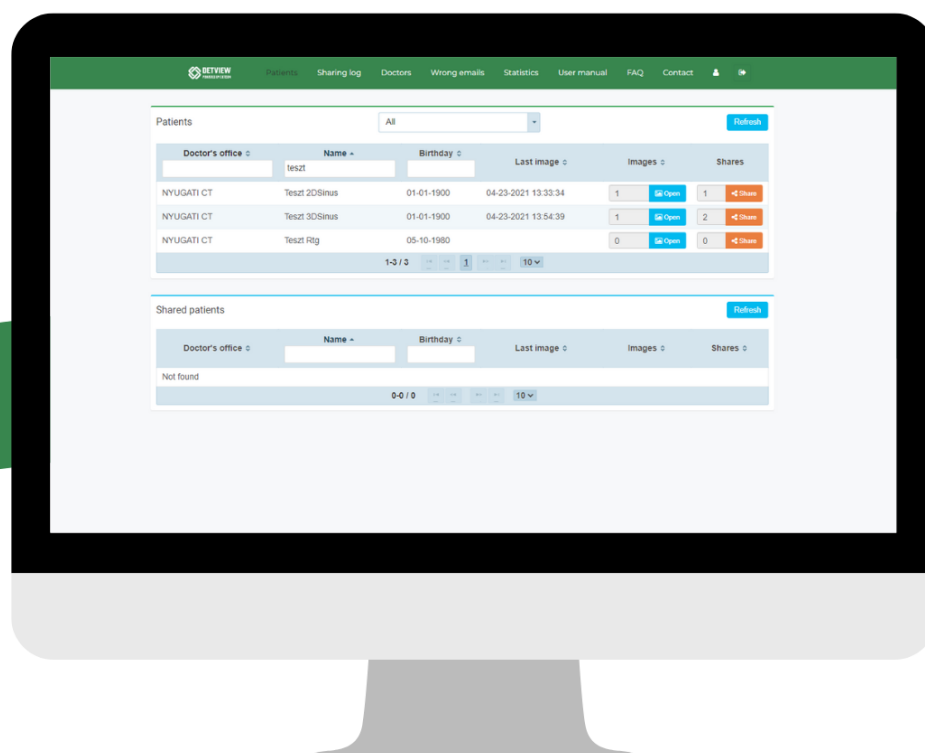

DetCom Systems Ltd.

Version: 4.0

© 2024

Content

Content	2
Introduction	2
Login page, Login	2
Patients Page	3
Profil page	7

Introduction


The document contains a detailed description of the admin level of the DetView system.

Login page, Login


There are additional language options available on the site, which can be selected from the drop-down menu.

Detailed description of the login:


1, You receive a message with accept link. If you clicking on this link, you agree to store your email address in the DetView system.

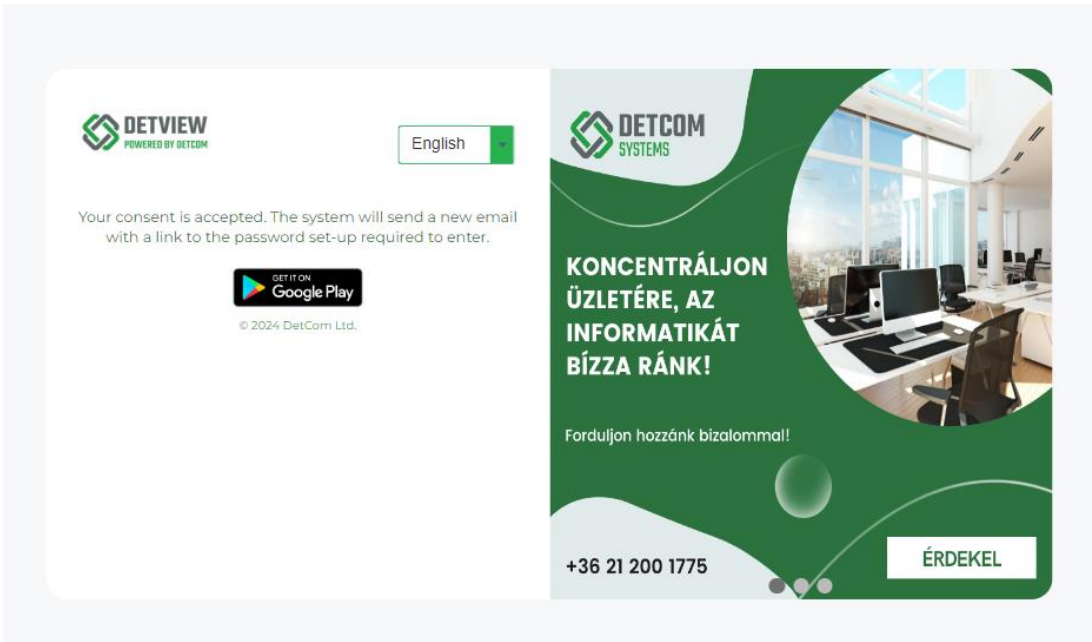

2, In the case of successful, you receive a new message with password setting link. Where you can enter own password. When you choose your password you should consider security, so it is advisable to choose a password in the following way: minimum 8 characters, including special characters, numbers, and capitalization.

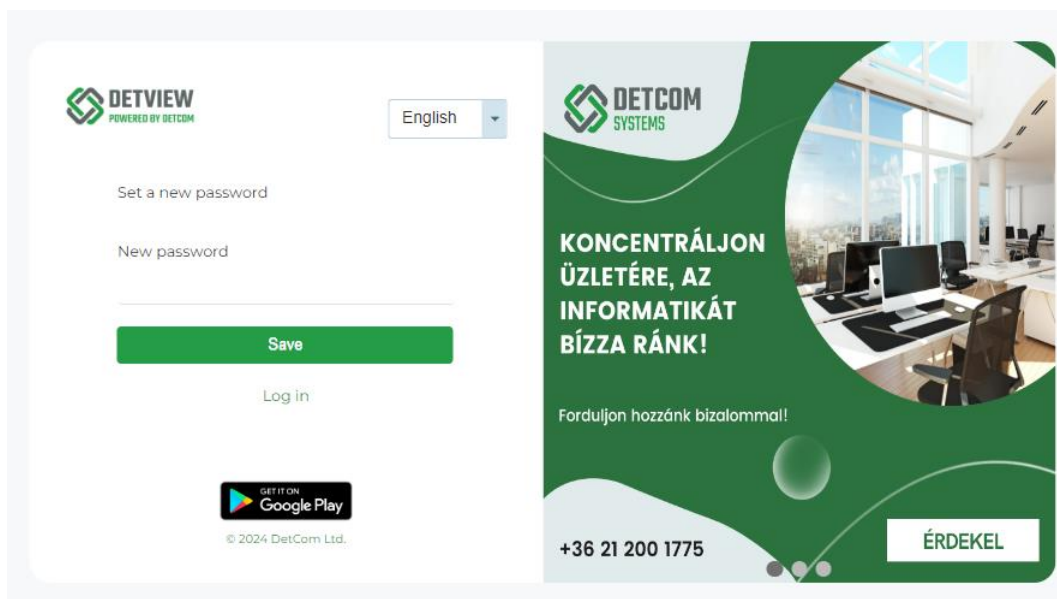

3, In the case of successful save, you receive email.

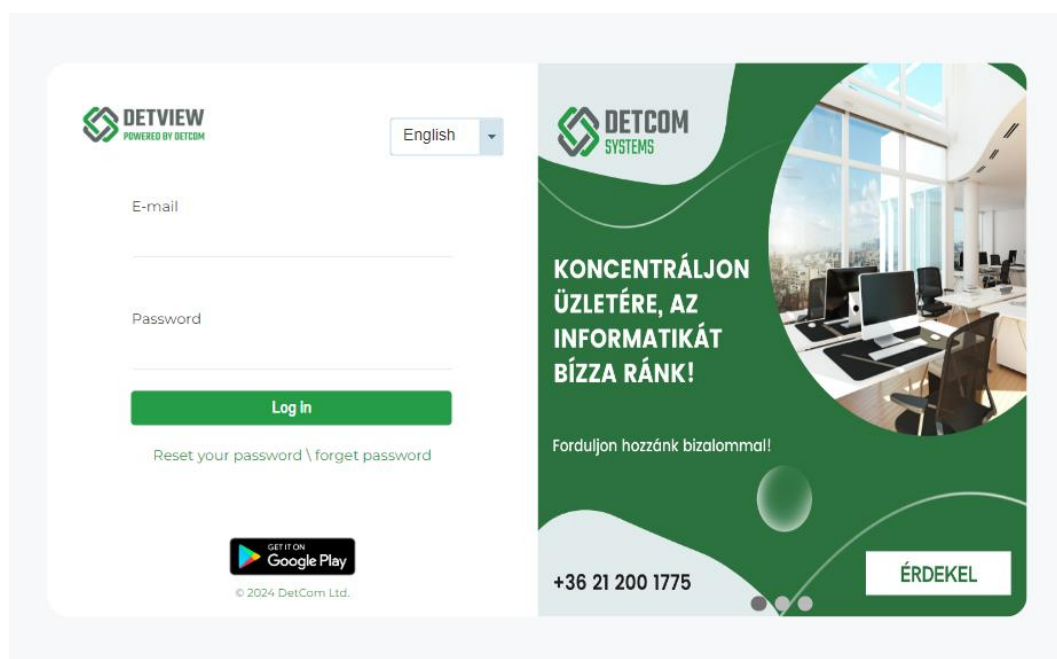













4, Then you get your main login surface. Enter your e-mail address and password here and click on "Sign In"

Patients Page

On this page, you can download 2D and 3D images of patients assigned to the user. Here you also have the opportunity to share the patient's images with other doctors and patients.

From the top menu, you can access the User manual and if you need add-ons for the application, you can download them from here.



Patients						Refresh	
All							
Doctor's office	Name	Birthday	Last image	Images	Shares		
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 	0 		
Cdent	DemoX-ray 2	03-19-2003	05-17-2024 11:21:36	122 	0 		
NFRC	DemoX-ray 2	03-19-2003	08-07-2024 14:58:34	567 	0 		
NFRC	Patient(en)information 1	03-20-2003	08-08-2024 14:10:38	1914 	0 		
NFRC	teszt teszt	05-10-1980	10-18-2023 12:58:49	4 	0 		
1-5 / 5						10	

Shared patients						Refresh	
Doctor's office	Name	Birthday	Last image	Images	Shares		
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 	0 		
1-1 / 1						10	

Detailed description of the page:

There are two sections on the page, one with the patients and one with the shared patients. The latter show the patients who are shared with other doctors or patients

- Columns on the surface can be sorted as desired.
- Filling the "**Name**" and the "**Date of Birth**" columns can result in faster searches.
- By clicking on the "**Open**" button:

Patients						Refresh	
All							
Doctor's office	Name	Birthday	Last image	Images	Shares		
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51 	0 		

we get all the images of the patient that can be retrieved from the system:

◀ Back to the patients

Demo Patient (12-05-1920) Refresh

Time ↕	Image type ↕	Teeth ↕	
01-14-2020 15:07:06	Intra recording		Dcm file download Image download Edit
01-16-2020 13:23:28	Intra recording	11	Dcm file download Image download Edit
01-17-2020 08:36:32	Intra recording	26	Dcm file download Image download Edit
01-20-2020 08:46:28	Intra recording	23	Dcm file download Image download Edit

There are several ways to download recordings on this page:

1. Clicking on the "**Image download**" button will immediately download the image in "jpg" format.
2. Clicking on the "**Edit**" button will display the recording, which can be armed as desired before downloading.



3. After clicking on the "**Dcm file download**" button, the recording will be downloaded in "dcm" DICOM format, so we can load it into our own SIDEXIS or other X-ray viewing systems.
4. After clicking on the "**3D without Viewer**" button, the images are downloaded in DICOM "folder" format, which can also be loaded into other compatible X-ray viewing systems

- Under the "**Download 3D with Sidexis4 Viewer**" button, you can download the format that can also be written to a DVD, which also includes the X-ray viewer. (This file is the largest, more than 1GB, so the download is longer!)

If we return to the previous page:

Patients						All		Refresh
Doctor's office	Name	Birthday	Last image	Images	Shares			
NFRC	Demo Patient	12-05-1920	05-16-2024 13:26:53	51	0	Open	Share	

By clicking on the "**Share**" button, you can share the images of the patient with a doctor or patient.

< Back to the patient

Patient share

Patient name you want to share
 Csabáné -29-1967)

Sharing email address
 Email
 Add

Doctors with sharing permission
 drnagy@kkt.com Delete add

Save

- Here enter the email address of the doctor or patient with whom we want to share the images.

First, select from the drop-down menu to share with a Patient or Doctor and enter the email address. After clicking on the "**Add**" button, a pop-up window will appear, where you will be informed about the data protection contribution. If you are impressed, click the "**Save**" button.

[Back to the patients](#)

Patient share

Patient name you want to share
 Demo Patient (01-01-1900)

Sharing email address
 Doctor [Add](#)

Shared License

[Save](#)

Attention

You are only entitled to share the X-ray image material and personal data associated with it if permitted by law! [Details](#)

☒ I read, I accept [Accept](#)

However, we also have the option for deleting added email addresses by clicking on the "Delete add" then "Save" button. In both cases, the system sends a notification notice to the given Email addresses.

- By default, only 10 patients and 10 images can be seen on the pages but it can be increased to 50 by using the drop-down menu below. If you would like to see more, you can turn a page on the bottom of the page.
- By clicking on "Sign out", which is on the right, you can log out of the system.

Profil page

In the profile menu you have the option to user to change their password or edit their details, including manage individual settings.

Password change

Old password

New password

New password again

[Save](#)

Personal data

E-mail address: detviewtest@detcom.hu

Name

☒ I don't want to be notified about new products, promotions and gift coupons.

[Edit data](#)

You can initiate the permanent deletion of your user account by sending a message to helpdesk@detcom.hu.